Polymerization unit

LONEC ZA POLIMERIZACIJO / LONAC ZA POLIMERIZACIJU
POLYMERISATIONSDRUCKTOPF / ПОЛИМЕРИЗАТОР

INTERDENT d.o.o.

Opekarniška cesta 26 · SI - 3000 CELJE · SLOVENIA

T: +386 (0)3 425-62-00 · E: info@interdent.cc · www.interdent.cc

Facebook: Interdent Slovenia

1	Potek dela · Upute za rad · Instruction for work Arbeitsverfahren · Процесс работы	3, 4, 5
2	Izjava o skladnosti · Izjava o sukladnosti Declaration of conformity · Konformitätserklärung Декларация о соответствии	6
3	Garancijski list za artikel 0603 · Jamstveni list za artikl 0603 Warranty for product ref. 0603 · Garantie für artikel 0603 Гарантия на артикул 0603	7, 8

Potek dela

Z dvigom ročaja sprostimo pokrov lonca. Pokrov zasučemo za 90° v levo ali desno in na ta način pokrov odstranimo z lonca. V lonec postavimo objekt, ki ga bomo polimerizirali in ga zalijemo s toplo vodo. Lonec zapremo in preverimo, če je izpustni ventil (4) zaprt. Nato preko priključnega ventila (6) doziramo zrak in na manometru (3) kontroliramo tlak v loncu. Pri tem vedno upoštevamo navodila proizvajalca akrilatnih mas. Ko je polimerizacija zaključena (čas polimerizacije določa proizvajalec akrilatnih mas), najprej sprostimo izpustni ventil (4). Ko lonec ni več pod pritiskom, lahko pokrov odpremo. Postopek je s tem zaključen. Maksimalni pritisk je 2,5 bara.

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Instruction for work

Release the lid by lifting the handle. Turn the lid by 90° clockwise or counterclockwise and thus remove it. Put the object to be polymerized into the pot and fill the pot with warm water. Close the lid and check if the release valve (4) is closed. Through the filling valve (6) fill the pot with air. Check the air pressure on the manometer (3). Always follow the instructions of the producer of acrylate material. When the polymerization procedure is over (the time of polymerization is prescribed by the producer of the acrylate material), release the air from the pot through the release valve (4). When all the air is released and the pot is no longer under pressure, you can open the lid. The polymerization procedure is now completed. The maximum air pressure is 2,5 bar.

After using the polymerization unit the pressure gauge, safety valve, discharge valve, and the air intake valve must be well cleaned.

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Signature of the seller:

Warranty declaration:

- ▶ The product will work flawlessly during the warranty period if being administered in accordance with its purpose and the instructions provided;
- ▶ Warranty period expires after 12 months from purchase day. At your request we will repair failure or defects on the product, if you will report it within the warranty period. Failure will be resolved free of charge within 45 days from the date of failure report. The product, which will not be repaired within this deadline, will be, at your request, replaced with a new one. We will extend the warranty period for the time the product was being repaired..
- ▶ Validity of the warranty should be proofed by certified warranty card and original invoice.

The warranty is voided:

- ▶ If an unauthorized person interferes into the device
- ▶ If provided instructions for use are not being considered
- ▶ If the device is being mechanically injured
- ▶ If the unoriginal spare parts are being build in

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