Thermotop

Aparat za potopno metodo / Device for dipping method

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Device for dipping method

DESCRIPTION:

Thermotop is an electrical device with adjustable heating temperature of the container intended for dipping wax. The device is meant to be used by dental technicians for cap making method in the dental laboratory. In combination with suitable wax extremely good and precise results can be achieved.

The wax can be melted without the change of the property of matter by having a certain amount of wax put into the Thermotop and setting the temperature.

High heat conductivity, long lifespan of the product, and ergonomic design.

- Previous working status and set values are memorized even when the power is off and on again as memory chip is built-in.
- Celsius and Fahrenheit (°C/°F) can be conveniently selected.

INSTRUCITONS FOR USE:

1. Plug in the Thermotop.

2. Turn on the power switch.

3. Press the key to select either Celsius or Fahrenheit temperature scale. (°C/°F to be displayed on the display window).

4. Then use the up and down arrow keys to set the desired temperature scale. ("S" LED light on).

Press the key to indicate the current temperature of the pot. ("C" LED light on)

5. Pressing the key once increases or decreases 1 degree of temperature (Continuing to press the arrow keys for 1-2 seconds makes the desired temperature set faster).

6. When the Thermotop is heating up to the set temperature the LED light is flickering. When it reaches the set temperature the LED light turns off.

SAFETY PRECAUTIONS

1. Do not touch the product with wet hands.
2. Precise circuit is built in this product. Excessive power or shock to the product can cause a failure. It is necessary to avoid giving impact which may cause mechanical trouble.
3. Be careful while using because the container is very hot and there is a danger of burns.
4. Do not use damaged power cord or plug, loose outlet (there is a danger of electric shock, fire).
5. Do not bend the power cord excessively or let it be squashed by a heavy object (there is a danger of electric shock, fire).
6. Plug must be connected to the outlet that has electrical grounding (there is a danger of electric shock, fire).
7. Do not block the ventilator located at the bottom of the product (it can cause overheat, fire).
8. Unplug the cord if lightning or thunder occurs, and when it is not used for a long time (there is a danger of electric shock, fire).
9. Do not pull out the cord or touch the power plug with wet hands (there is a danger of electric shock, fire).
10. Do not install near moisture or dust, or a place where water splashes (there is a danger of electric shock, fire).
11. Do not install near electric heating appliances (there is a danger of fire).
12. Unplug the power cord, and do not directly spray water or wipe it with chemicals etc. (there is a danger of electric shock, fire).
13. Keep power plug clean, free of dust and water. (there is a danger of electric shock, fire).
14. If you detect noise or smell smoke coming from the unit, unplug it immediately (there is a danger of electric shock, fire).
15. Do not disassemble, repair, remodel carelessly (there is a danger of electric shock).
16. Contact us with any questions or detailed information about troubleshooting.

* Manufacturer will never be responsible for any problems occurred by the misuse of the product in a manner other than prescribed above.

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We Interdent d.o.o., Opekarniška cesta 26, SI – 3000 Celje declare that products below are in accordance with EU directives

Naziv izdelka: Aparat za potopno metodo
Name of the product: Device for dipping method

Tip: THERMOTOP
Type: THERMOTOP

Upoštevane EU smernice:

EU directives:

- EU – nizkonapetostna smernica: 2014/35/EU
EU – low voltage directive: 2014/35/EU
- EU – smernica za elektromagnetno združljivost: 2014/30/EU
EU – directive for electromagnetic compatibility: 2014/30/EU

Testne metode po standardih:

Test methods according standards

- EN 61326-1:2013
- EN 61000-3-2:2014
- EN 61000-3-3:2013
- EN 61010-1:2010

Standard kakovosti: ISO 9001: 2015

Quality standard: ISO 9001:2015

Proizvod je varen za uporabo v standardnih pogojih. Vsaka sprememba proizvoda, ki ni odobrena s strani proizvajalca, razveljavlja to izjavo.

Product is safe in standard conditions. Any change of the product that is not approved by producer, repeals this declaration.

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- Izdelek bo v garancijskem roku brezhibno deloval, če ga boste uporabljali v skladu z njegovim namenom in priloženimi navodili. / *The product will work flawless during the warranty period if being administered in accordance with its purpose and the instructions provided.*
- Garancijska doba traja 12 mesecev od dneva prodaje. Na vašo zahtevo bomo popravili okvaro in pomanjkljivosti na proizvodu, če boste le-to sporočili v garancijskem roku. Okvaro bomo brezplačno opravili najkasneje v 45 dneh od dneva prijave okvare. Proizvod, ki ga ne bomo popravili v omenjenem roku, bomo na vašo zahtevo zamenjali z novim. Za čas popravila vam bomo podaljšali garancijski rok. / *Warranty period expires after 12 months from purchase day. At your request we will repair failure or defects on the product, if you will report it within the warranty period. Failure will be resolved free of charge within 45 days from the date of failure report. The product, which will not be repaired within this deadline, will be, at your request, replaced with a new one. We will extend the warranty period for the time the product was being repaired.*
- Veljavnost garancije dokazujete s potrjenim garancijskim listom in original računom. / *Validity of the warranty should be proofed by certified warranty card and original invoice.*

Garancija preneha veljati / The warranty expires :

Če v aparat posega nepooblaščen oseba. / *If an unauthorized person interferes into the device.*

Če niso upoštevana priložena navodila za uporabo. / *If provided instructions for use are not being considered.*

Če je izdelek mehansko poškodovan. / *If the device is being mechanically injured.*

Če so vgrajeni neoriginalni rezervni deli. / *If the unoriginal spare parts are being build in.*

INTERDENT d.o.o.

Opekarniška cesta 26 · SI - 3000 CELJE · SLOVENIA

T: +386 (0)3 425-62-00 · E: info@interdent.cc · www.interdent.cc

Facebook: Interdent Slovenia